


Experimental studies

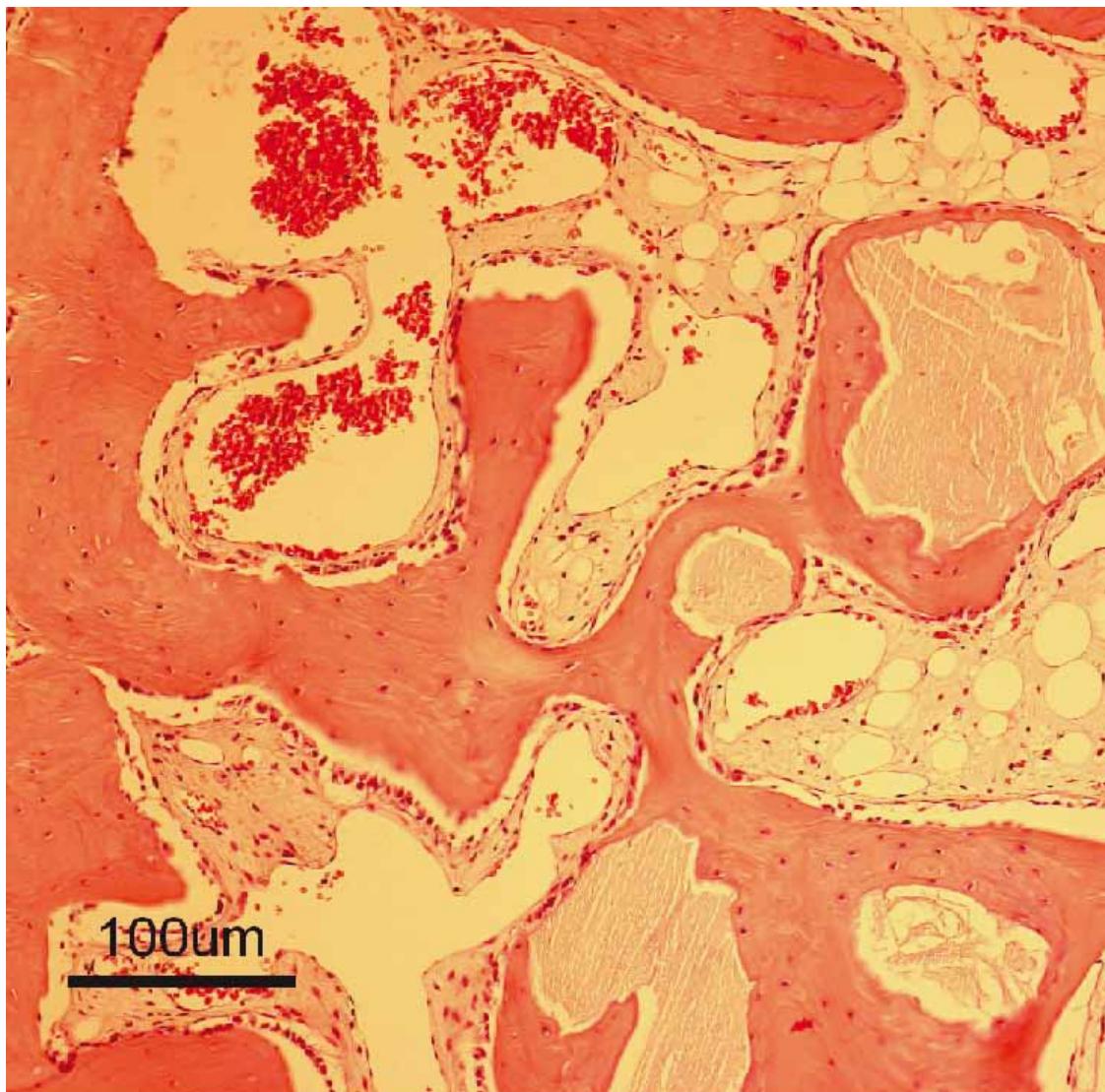

OsteoBiol®
by Tecnoss

REGENERATION SCIENCE

INSPIRED BY NATURE



EXPERIMENTAL STUDIES

098

A Scarano¹
F Lorusso²
L Ravera²
C Mortellaro³
A Piattelli²

Bone regeneration in iliac crestal defects: an experimental study on sheep

ABSTRACT

Successful implant placement requires adequate alveolar ridge dimensions and, if the implant site presents a lack of bone, Guided Bone Regeneration (GBR) is the surgical procedure commonly performed in order to provide an augmentation in terms of volume for the insertion of dental implants. Several types of membranes and biomaterials have been proposed for GBR techniques and the selection of the most appropriate grafting material is one of the key factors in achieving adequate bone formation.

The aim of the present study was to determine the *in vivo* tissue responses and gap healing patterns around dental implants treated with cortico-cancellous porcine bone blocks, collagenated cortico-cancellous porcine bone versus only membrane in a standardized sheep peri-implant gap-defect model. In the iliac crest of six sheep 4 defects were created for the insertion of an implant and the defects were filled with 1) control, only membrane (OsteoBiol® Evolution, Tecnoss®, Giaveno, Italy); 2) 250–1000 µm cortico-cancellous particulate porcine bone mix (OsteoBiol® Gen-Os®, Tecnoss®) + resorbable equine pericardium membrane (OsteoBiol® Evolution) (test 1); 3) cancellous equine bone blocks (OsteoBiol® Sp-Block, Tecnoss®) + resorbable membrane (OsteoBiol® Evolution) (test 2); 4) pre-hydrated collagenated cortico-cancellous porcine bone mix (90% granulated mix, 10% collagen gel) (OsteoBiol® mp3®, Tecnoss®) + membrane (OsteoBiol® Evolution) (test 3). The animals were sacrificed after a 4-month healing period and all specimens were processed and analyzed with histomorphometry, with the result that all experimental groups showed an increase of new bone. From the findings it is evident that particles of cortico-cancellous porcine bone 250–1000 µm particulate mix (CCPB) favour bone formation with a result similar to those obtained with pre-hydrated collagenated cortico-cancellous porcine bone mix (PCCPB). All biomaterials used in the present study were characterized by the presence of bone formation and absence of inflammatory cell infiltrates. However, the defect treated by membrane alone was characterized by the presence of soft tissues and a little immature bone.

CONCLUSIONS

As stated by the Authors, “the function of the graft is not only to improve the space-making capabilities of the membrane, but also to provide additional points on which osteoblasts can start forming new bone. We have shown that CCPB and PCCPB promote bone regeneration in large defects (7 mm wide and 4 mm deep) around dental implants”.

In conclusion, this study demonstrates that particulate porcine bone mix and porcine cortico-cancellous collagenated pre-hydrated bone mix, used as scaffolds, induce bone regeneration and these findings suggest that these biomaterials are characterized by a high biocompatibility and can induce a faster and greater bone formation.

Grafted with

OsteoBiol® Gen-Os®
OsteoBiol® mp3®
OsteoBiol® Sp-Block

MEMBRANE

OsteoBiol® Evolution